extracting the essence.

extracting the essence.

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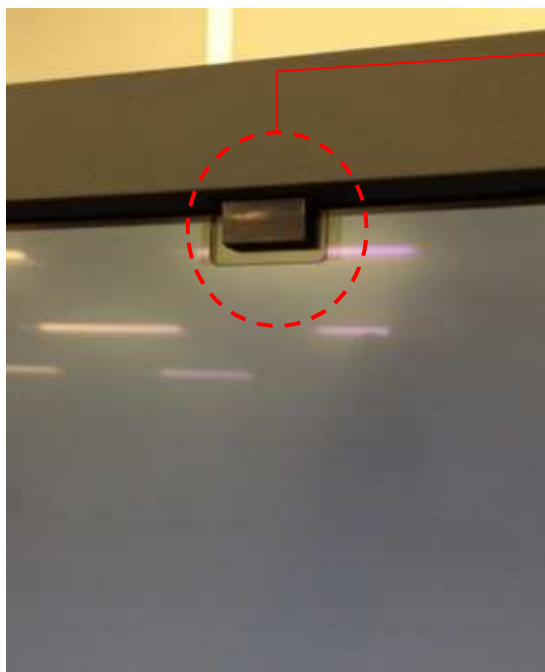
DisplayPort w/ 10bit

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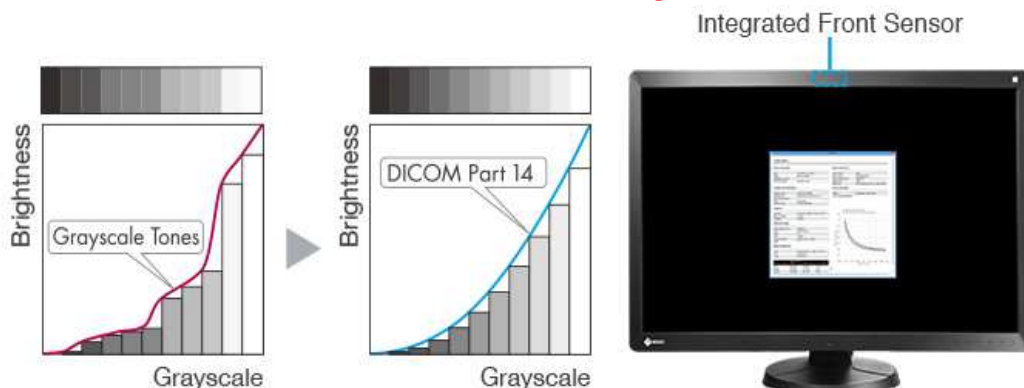
Presense Sensor


DICOM - Digital Imaging and Communications in Medicine

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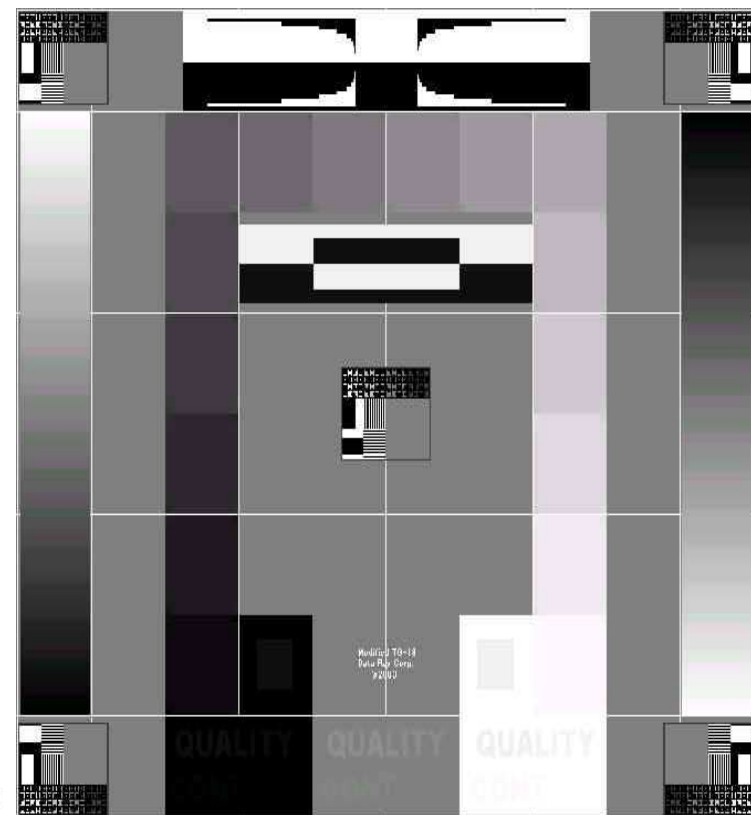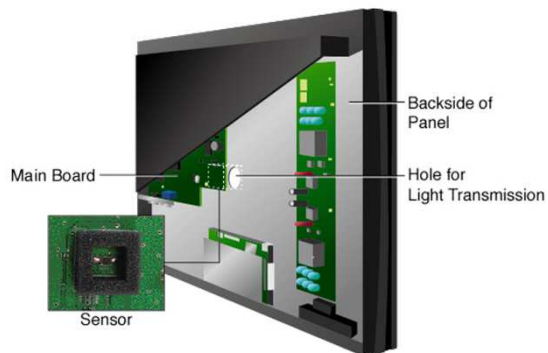
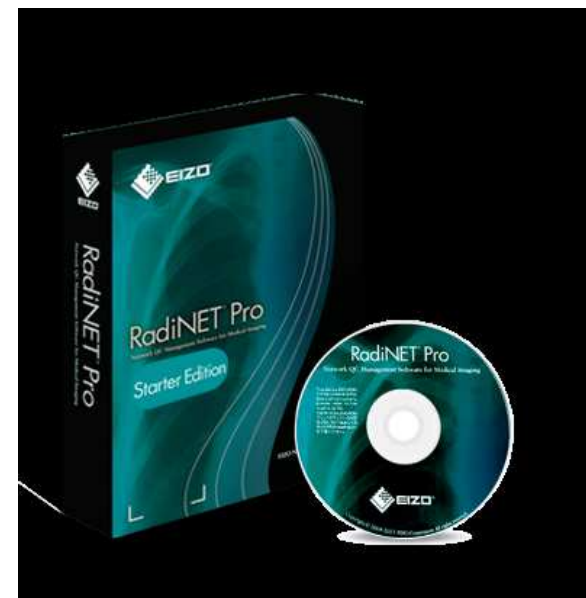
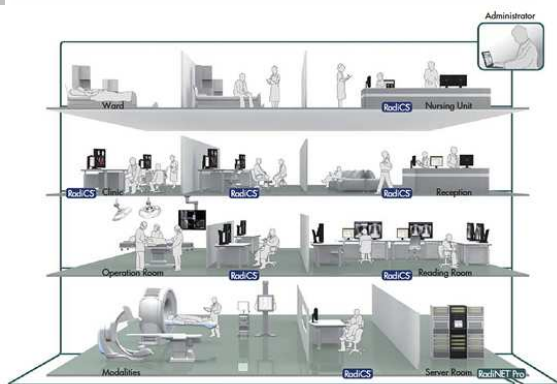
There are some monitors that should be confirmed the status. [Confirm...](#)

Filter

Location

⊕ All Monitors

Keyword

Alerted Monitors

Monitor Details

Customize

Filtering : All Monitors [Reset](#)

Computer : 2, Monitor : 6, Altered Monitor : 4

View : Simple View ▾

Sort : Judgment Date ▾ Judgment Display Task : Latest Result ▾

[Display Item Settings](#)

Within 1 month ⌆

East3F > Roentgenology > Radiogram Room
XXX-XXX
RX210 00000001 - DICOM

OK 1400H

✓ Constasy Test : 06/05/2012 Passed

Connected
Calibration is completed.

⚠ Schedule Reminder (Next Constasy Test : 07/15/2012),
Remote Operation Error, Error

West1F > Item 1.2.2 > Item 1.2.2.2
YYY-YYY
RX340 00001111 - DICOM

OK 62H

✓ Constasy Test : 06/04/2012 Passed

Connected

East4F > AAA > bbb
YYY-YYY
MX270W 10017777 - DICOM

OK 11260H

✓ Constasy Test : 06/04/2012 Passed


Connected
Executing Calibration

Operation ▾

Copyright © 2012 EIZO NANA CORPORATION All Rights reserved

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	AAPM TG-18	DIN V 6868- 57	QS-RL	IPEM	IEC	JESRA
Established in						
Established by	American Association of Physicists in Medicine (AAPM)	Deutsches Institut für Normung e. V. (DIN)	Qualitätssicherungs-Richtlinie (QS-RL)	Institute of Physics and Engineering in Medicine (IPEM)	International Electrotechnical Commission (IEC)	Japan Industries Association of Radiological Systems (JIRA)
Acceptance Test	Yes	Yes	Yes			Yes
Constancy Test	Yes		Yes	Yes	Yes	Yes
Category for Diagnostic Monitor	Primary	Category A (For X-ray)		Yes		Grade 1
Category for Review Monitor	Secondary	Category B (For CT, MR, Dental Equipment)				Grade 2
RadiCS Support	 Yes	Yes	Yes	Yes	Yes	Yes

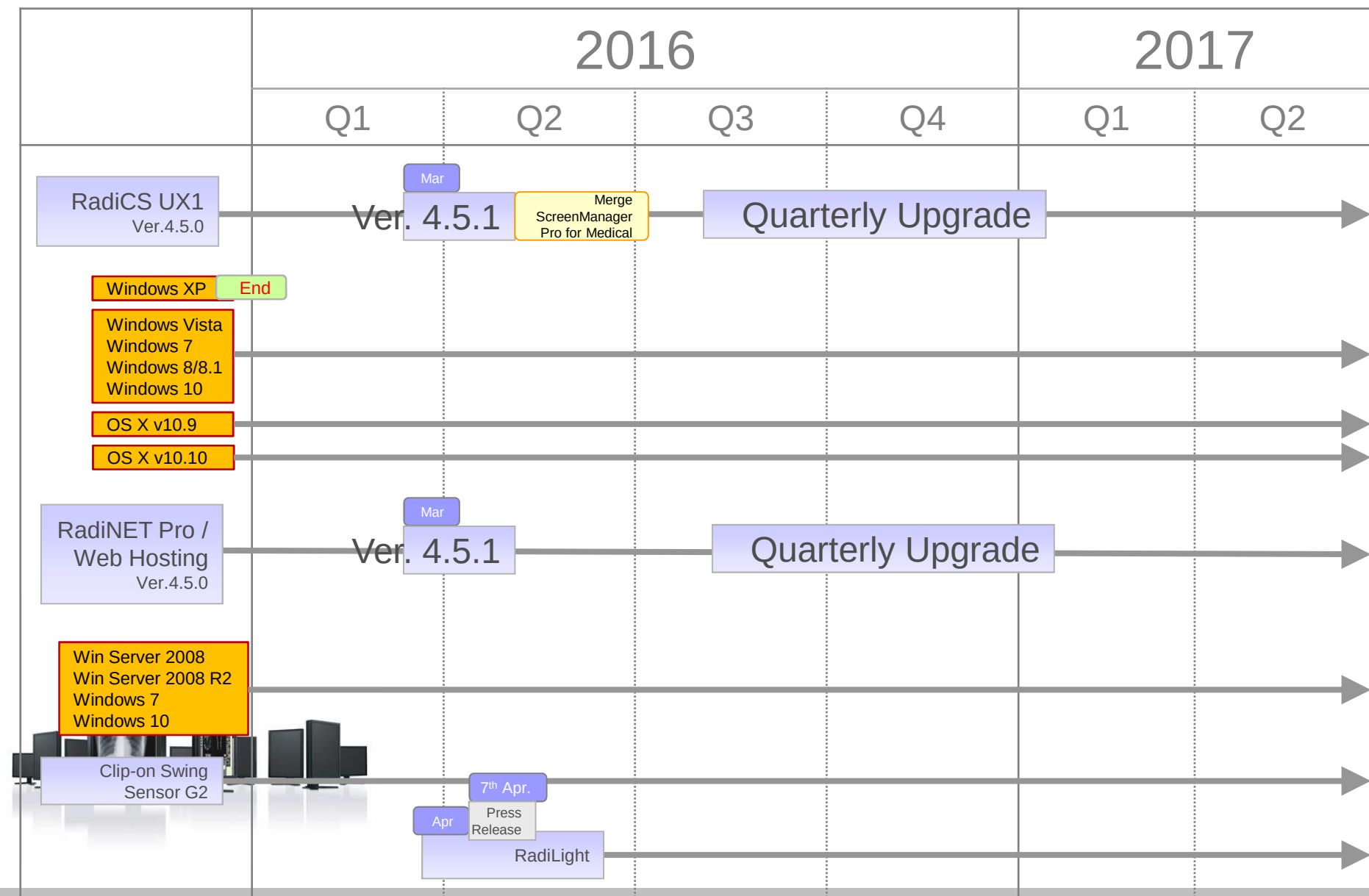Test

Acceptance Test
Visual Check
Consistency Test(Every Month/Quarter)
Consistency Test(Annual)

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The screenshot shows the RadiCS software interface, Version 4.2.1. The interface includes a menu bar with 'Main Menu', 'Monitor List', 'Report Archive', and 'Others'. A search bar is present with a dropdown menu showing 'All', 'Failed', and two specific monitor models: 'EIZO GX540-CL 24760094' and 'EIZO GX540-CL 24767094'. Below the search bar is a table of test results.

Date / Time	Task	Judgment	QC Guideline	Tester	Monitor	CAL Mode
01/27/2015 14:58	Calibration	Passed	-	Administrator	EIZO GX540-CL 24760094	DICOM
01/27/2015 14:57	Calibration	Passed	-	Administrator	EIZO GX540-CL 24767094	DICOM
01/27/2015 14:48	Hands-off Check	Passed	-	RadiCS	EIZO GX540-CL 24767094	DICOM
01/27/2015 14:47	Hands-off Check	Passed	-	RadiCS	EIZO GX540-CL 24760094	DICOM
01/27/2015 14:44	Grayscale Check	Passed	-	Administrator	EIZO GX540-CL 24760094	DICOM
01/27/2015 14:44	Grayscale Check	Passed	-	Administrator	EIZO GX540-CL 24767094	DICOM
01/27/2015 14:42	Luminance Check	Passed	-	Administrator	EIZO GX540-CL 24760094	DICOM
01/27/2015 14:42	Luminance Check	Passed	-	Administrator	EIZO GX540-CL 24767094	DICOM
01/27/2015 14:41	Daily Test	Passed	Basic Mammo QC	Administrator	EIZO GX540-CL 24767094	DICOM
01/27/2015 14:41	Daily Test	Passed	Basic Mammo QC	Administrator	EIZO GX540-CL 24760094	DICOM

At the bottom of the interface, there is a status bar indicating 'UX1 Sensor is connected.' and buttons for 'Multiple Report' and 'Report'.

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ICS RadiCS

Version 4.2.1
[About RadiCS](#) RadiCS®

Main Menu | Monitor List | Report Archive | Others ▼

Registration Information

Schedule

RadiCS SelfQC

Sensor

Password

RadiNET Pro

User Mode

☐ Activate scheduler

☒ Visual Check	Daily 00:00	Change...
☒ Constancy Test	Quarter 1st(Jan.-routine) 00:00 Execute test	Change...
☒ Calibration	Quarter 1st(Jan.-routine) 00:00	Change...
☒ Hands-off Check	Daily 00:00	Change...

Policy

☒ On schedule
☐ At login

☒ Stops the process when a person is detected.

* A presence sensor is required.
* Applicable only to task execution using the Swing Sensor/Integrated Front Sensor.

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